

Intended use

Vivresse is a gynaecological cream with a barrier effect that helps to restore the physiological pH of the vagina and allows the recovery of optimal conditions for the proliferation of naturally-occurring lactobacilli. Therefore, avoiding the colonisation and overgrowth of pathogens such as *Candida albicans*, *Gardnerella* and others that cause recurrent vaginal infections. Vivresse alleviates signs and symptoms usually present in these circumstances such as redness, itching, discharge and atrophy. Vivresse can be recommended as a symptomatic adjuvant to antifungal treatments as well as a barrier cream useful to prevent recurrent vaginal infections.

Presentation

7 tubes containing 5 ml.


Composition

Pea protein and grape seed extract, lactic acid, squalene, purified water, sodium acrylates copolymer, Paraffin, phenoxyethanol, trideceth-polypropyleneglycol, sorbitan oleate, Piroctone olamine, Maltodextrin.

Instructions for use

Apply Vivresse in the supine position at bed time directly into the vagina with the aid of the applicator provided.

- Lie on your back with your knees drawn up.
- Gently insert the applicator deep into the vagina.
- Push the plunger to fully dispense the cream.
- Remove the applicator from the vagina.

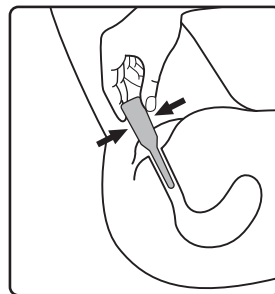

Dosage

One application daily until symptoms disappear.

Warnings

- A vaginal infection must be diagnosed by a healthcare professional. We therefore recommend you consult your doctor if you have any doubts on the diagnosis, or if you have severe symptoms, are under 14 years or have any other diseases or risk factors.
- A vaginal infection may require a pharmacological treatment with an antibiotic or antimycotic substance. This device is not an antibiotic nor an antimycotic, as it does not have a bactericidal, bacteriostatic or antifungal effect against the pathogens and, therefore, it will not substitute any such treatment recommended by a healthcare professional. When a pharmacological treatment is needed, this device can be used concomitantly.
- Do not use this product during pregnancy or breastfeeding. Consult your doctor first.
- If you develop an allergic reaction while using the device stop application immediately and consult a doctor.
- Do not use the product after the expiry date.
- Do not use the product if damaged
- Do not freeze.
- Keep away from heat and light sources.
- Keep the product out of the reach of children.

Contraindications

Do not use Vivresse in case of hypersensitivity to any of the ingredients contained in the device.



DEVINTEC SAGL
Corso Elvezia 14
6900 Lugano - Switzerland



Devintec Health S.L.
Calle General Pardiñas 99.6ª.
28006 Madrid – Spain

